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1 Please install the free **Soehnle Connect-app** from google play store (android™ smartphone / tablet) or from apple app store (iPhone® / iPad®).

2 Activate bluetooth on your smartphone or tablet.

Notice: The app is using an energy saving version of bluetooth. This version needs the permission of smartphone location. Soehnle is not using this information at all.

3 Start the Soehnle Connect-app on your smartphone.

Full data control by Soehnle: Please choose **“Register”** if you want to secure your data in the Soehnle cloud also, or you choose **“Skip registration”** if you want to use and keep your data locally on your phone only (fig. 1).

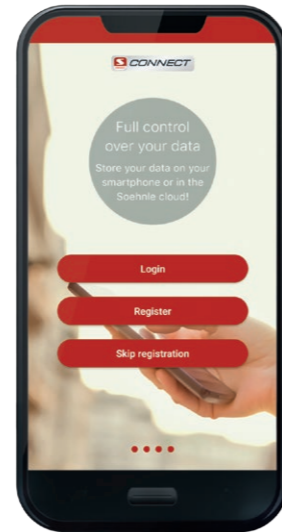
Fig. 1

4 **“Skip”:** Please enter now your name, gender and age [date of birth] (fig. 2).

“Register”: Please enter your email additionally and choose your individual password (fig. 3). Subsequently, select the device type scale (fig. 4) and the specific model, which you want to connect to the app (fig. 5).

Notice: The password specification requires a combination of alphabets and numbers, minimum 9 characters and one uppercase. Your input for gender, age (date of birth) and body height later on are necessary to determine body analysis values.

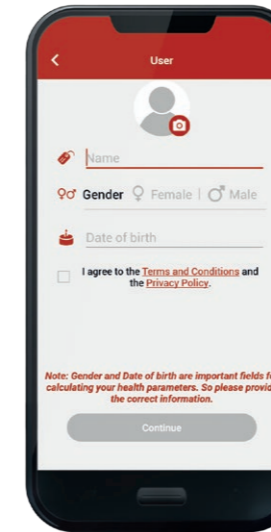
Fig. 2

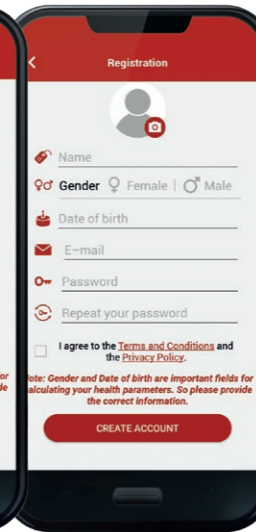
Fig. 3

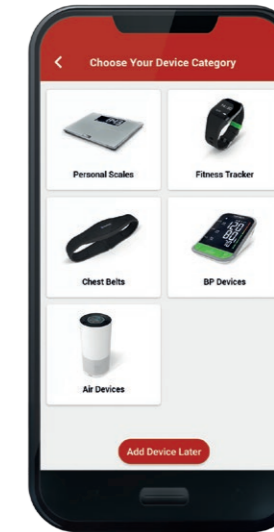
Fig. 4

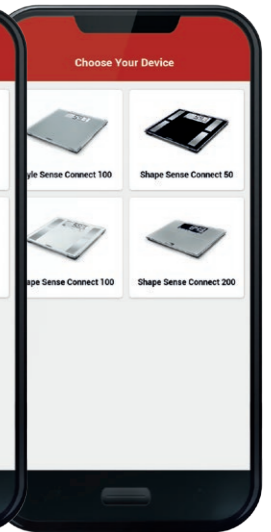
Fig. 5

5 **Almost there:** Now you are able to connect the app with the personal scale: Step with one foot shortly on the scale to turn it on (fig. 6).

The bluetooth symbol on the scale's display starts blinking. The app will now connect with your scale automatically.

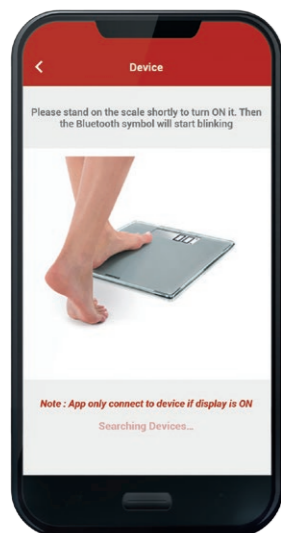
Fig. 6

6 **Here we go directly!** Please step on the scale barefoot for your first measuring. This may last up to 30 seconds. The app shows your weight first. At the same time, the body analysis will be processed in the background (fig. 7).

To transfer the measured data to the Soehnle Connect-app in future, please click on the arrow-symbol on the **homescreen** (fig. 8) or pull-down to refresh on the **dashboard** (fig. 9) direct after a measurement or at any later point of time.

Please be aware, that the scale needs to be turned on each time of synchronisation.

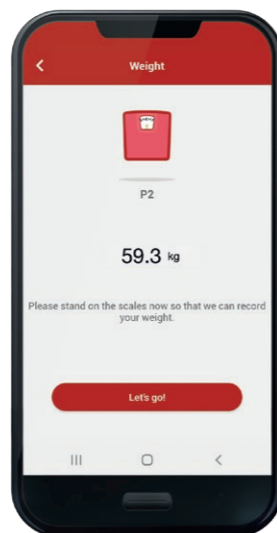
Fig. 7

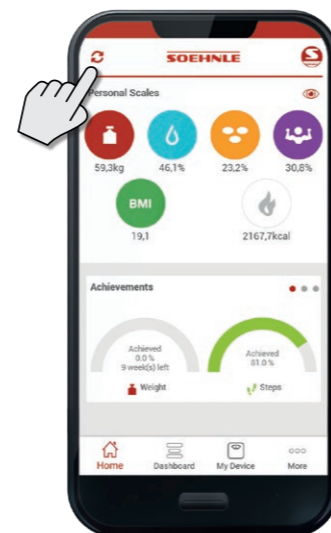
Fig. 8


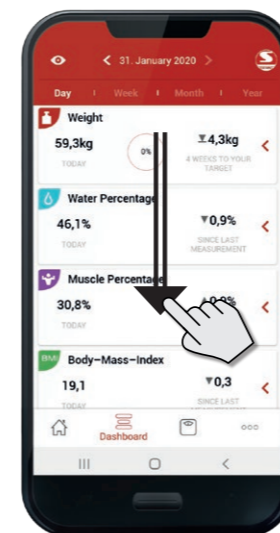
Fig. 9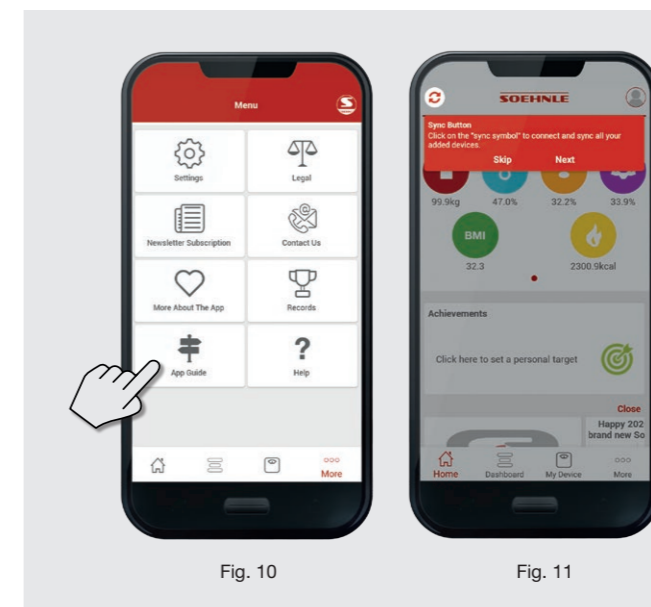

TIP

To get an easy guide through the Soehnle Connect-app please use the app guide in the **“More”** menu (fig. 10). That way, you may quickly get used to the most important features of the app (fig. 11).

Fig. 10

Fig. 11

Now have fun with your brand new quality connect scale from Soehnle!

Please always get the latest information on the Soehnle Connect-app, the smartphone compatibility list or frequently asked questions (FAQ) at soehnle.de/soehnle-connect