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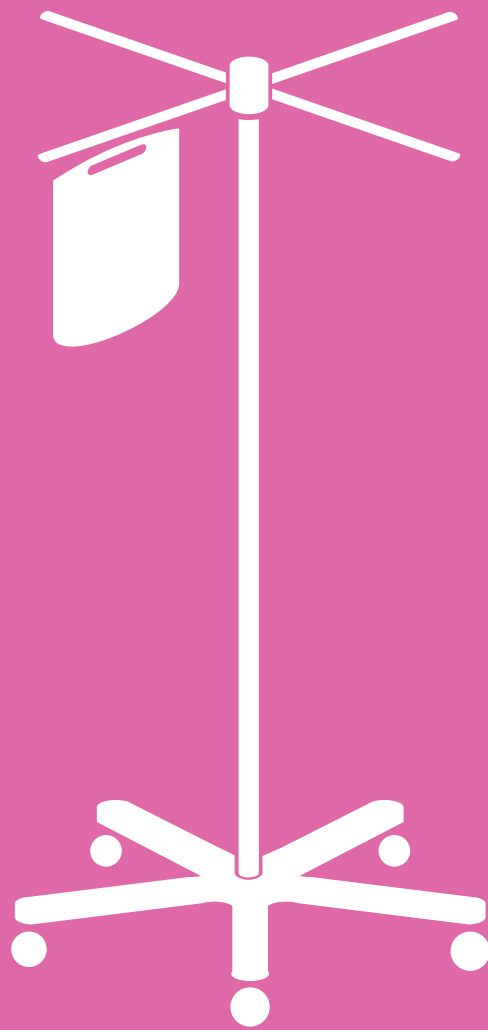
**INFUSIONSSTÄNDER
MEDI-CARE**

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To adjust the height of the extension tube loosen the hand wheel (15), raise or lower the extension tube (14a) to the desired height and tighten the hand wheel again.

An untimely disengaging of the hand wheel leads to sudden drop of the extension tube. – **Warning.**

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Push the drip glass bracket (6b) with your thumb on the marked spot rectangular onto the stand.

– **NOTE:** Tilting the bracket while pushing it, could lead to breakage!

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o.k.

This infusion stand is maintenance-free in regard to its mechanical function. However, at least every 6 months it is essential to check thoroughly the correct function of all screw connections and the faultless operational condition. The screw thread of the handwheels (15) must be greased from time to time. The covered inner pins/axles of the castors are made of galvanised steel. Should stronger protection against corrosion be required, possibly needing a different type of castors, then please contact us or your authorised supplier. Room temperatures below 7°C are liable to cause rust damage to the chromed parts.

All surfaces can be wiped clean with a dry cloth. If using a wet cloth all surfaces must be dry-rubbed most thoroughly.

CAUTION: Never use very wet cloths as no moisture must enter the structure or components of this infusion stand.

Never use abrasive chemical cleaners or scouring agents as these will damage the chromium parts and surfaces. Do not use concentrated or diluted iodine or hydrochloric acid as the fumes would be destructive to this infusion stand. Chrome products cannot replace stainless steel in hygienic locations. Well known base agents in disinfectants, if wrongly applied or mistakenly used, can lead to damage of surfaces.

For thorough disinfection we recommend the preparations of „Mikrozid sensitive Liquid“ and „Mikrozid sensitive Wipes“ from the Company Schülke & Mayr*. Care must be taken to wipe all agents from treated surfaces after application not longer than 1 minute as otherwise damage could occur, especially corrosion, which also affects stainless steel surfaces, due to the chloride compounds in the cleaning solution.

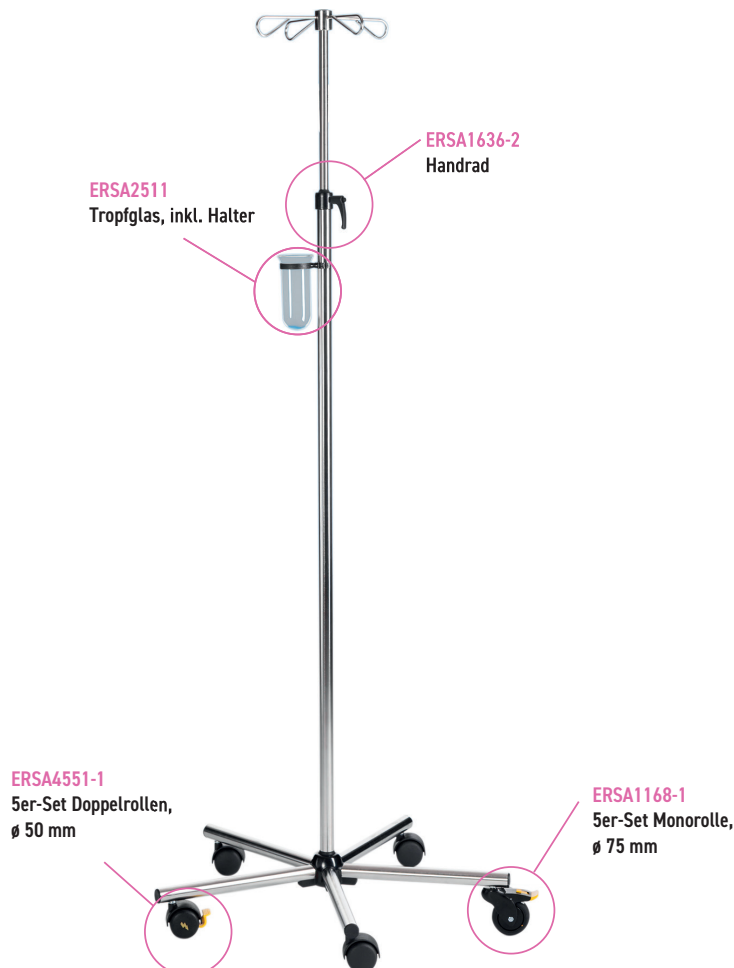
No solution is to be allowed to enter the inside parts as a high concentration of waste agent is likely to build up, eventually resulting in damages with loss of function of the mechanical components. For this reason the use of spray disinfectants is to be avoided absolutely. Please bear in mind that after each activation period all agents must be completely removed so as to avoid any damage to manifest itself at a future date. Should these recommendations be disregarded the useful life of this infusion stand could seriously be compromised.

The foregoing recommendations regarding disinfectants are for guidance only. Responsible for the care and maintenance of this infusion stand remains entirely with the user. In case of any doubt about the suitability of a given product, contact its manufacturer to confirm its particular qualities and to advise on the exact chemical formulation and/or corresponding changes that may have been made in the interim.

* Schülke & Mayr GmbH, Robert-Koch-Str. 2, D-22851 Norderstedt
Fax +49 40 521 00 - 318, www.schuelke.com

* Schülke & Mayr GmbH, Robert-Koch-Str. 2, D-22851 Norderstedt
Fax +49 40 521 00 - 318, www.schuelke.com

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Hersteller: Firma SIMPEX GmbH